


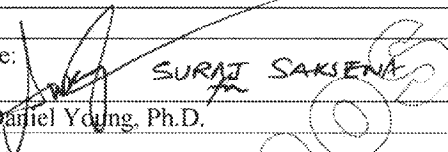
theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description		Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma	
Originator: Curtis Schneider		Date: 09/25/2013	

Validation of Modified Siemens Total Cholesterol Assay


Author(s):

Signature: 	Date: 11/21/13
Name: Paul Patel, Ph.D.	Title: Team Lead, General Chemistry

Reviewer(s):

Signature:	Date:
Name:	Title:
Signature:  SURAJ SAKJANI	Date: 11/7/13
Name: Daniel Young, Ph.D.	Title: Vice President

Approver(s):

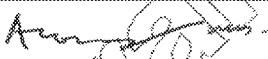Signature: 	Date: 11/17/13
Name: Adam Rosendorff, M.D.	Title: Laboratory Director

 9/19/13
 Sunil S. Dhawan M.D.

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

TMP-00009 Rev. A, Released 08/01/13

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider		Date: 09/25/2013	

Total Cholesterol Plasma Assay

- I. Overview**
- II. Method Principle**
- III. Definitions and Abbreviations**
- IV. Pre-clinical Validation**
 - a. Analytical Measurement Range
 - i. Limits of Blank, Detection and Quantitation
 - ii. Linearity
 - b. Analytical Specificity
 - c. Precision
- V. Clinical Validation**
 - a. Method Comparison with Predicate
 - b. Transference and Verification of Reference Interval (Venous)
 - c. Verification of Reference Interval with Finger Stick Samples
- VI. Stability**
 - a. Reagent
 - b. Sample
 - c. Calibrators

Theranos Internal Only

Theranos Confidential

Page 2 of 13

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

TMP-00009 Rev. A, Released 08/01/13

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider		Date: 09/25/2013	

I. Overview

Cholesterol is a steroid with a secondary hydroxyl group in the C3 position. It is synthesized in many types of tissue, but particularly in the liver and intestinal wall. Approximately 75% of cholesterol is newly synthesized and 25% originates from dietary intake. Normally, the cholesterol in the plasma or serum is 60%-80% esterified. Approximately 50-75% of the plasma cholesterol is transported by low density lipoproteins (LDL) and 15-40% by high density lipoproteins (HDL).


Serum cholesterol is elevated in hereditary hyperlipoproteinemias and in various other metabolic diseases. Moderate to elevated values are also seen in cholestatic liver disease. Hypercholesterolemia reflects an increase of lipoproteins of 1 or more specific classes (e.g., β -LDL, α -1 HDL, α -2 HDL, or LP-X). Hypercholesterolemia is a risk factor for cardiovascular disease.

Low levels of cholesterol can be seen in disorders that include hyperthyroidism, malabsorption, and deficiencies of apolipoproteins.

II. Method Principle

The cholesterol esters are hydrolyzed by cholesterol esterase to cholesterol and free fatty acids. The cholesterol is converted to cholest-4-en-3-one by cholesterol oxidase in the presence of oxygen to form hydrogen peroxide. A colored complex is formed from hydrogen peroxide, 4-aminoantipyrine and phenol under the catalytic influence of peroxidase. The absorbance of the complex is measured as an endpoint reaction at 505/694 nm.

Reaction Equation

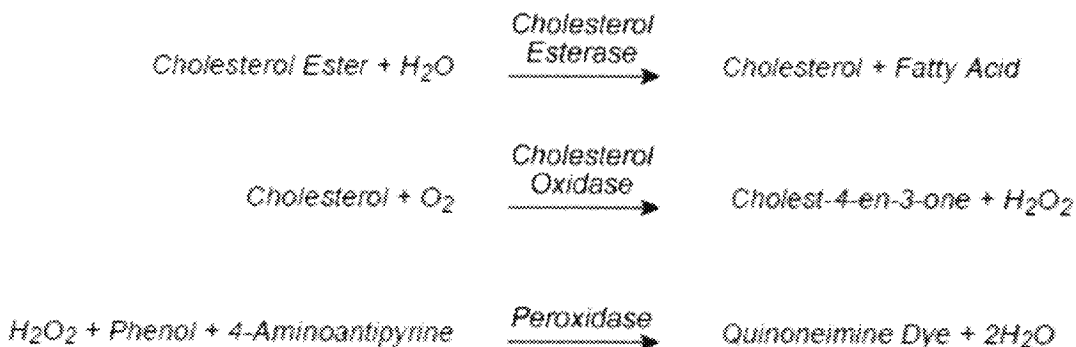

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider	Date: 09/25/2013		

III. Definitions and Abbreviations

The following definitions and abbreviations are used in this document and related documents and attachments:

- a. **Accuracy:** Accuracy is defined by CLSI as the closeness of agreement between a test result and an accepted reference value. Method accuracy is used in a different sense by the American Association of Pharmaceutical Scientists where it is expressed as percent relative error (%RE). Trueness, a related CLSI term, is the closeness of agreement between the average of a number of replicate measured quantity values and a reference quantity value.
- b. **Analyte:** Component represented in the name of a measurable quantity. The closely related term measurand is defined as the particular quantity subject to measurement.
- c. **Analytical sensitivity:** There are several alternative uses of this term. Most commonly, and for the purposes of this Validation Plan, it is used interchangeably with limit of detection. It is also used to describe the ability of an analytical method to assess small variations of the concentration of an analyte, such as the slope of the calibration curve (IUPAC).
- d. **Analytical specificity:** Ability of a test or procedure to correctly identify or quantify an entity, including in the presence of interfering substance(s) or phenomena.
- e. **Calibration:** Set of operations that establish, under specified conditions, the relationship between values of quantities indicated by a measuring instrument or measuring system, or values represented by a material measure or a reference material, and the corresponding values realized by standards. Under CLIA, calibration refers to the process of testing and adjusting an instrument, kit, or test system, to provide a known relationship between the measurement response and the value of the substance being measured by the test procedure (42 CFR 493.1217).
- f. **Calibrator:** A substance, material, or article intended to be used to establish the measurement relationships of a diagnostic medical device.
- g. **CLIA:** Clinical Laboratory Improvement Amendments of 1988. Congressional legislation that defined and requires specific quality assurance practices in clinical laboratories.
- h. **CLSI:** Clinical and Laboratory Standards Institute.

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider		Date: 09/25/2013	

- i. **Coefficient of Variation:** The ratio of the standard deviation to the average, often multiplied by 100 and expressed as a percentage, abbreviated as %CV.
- j. **Colorimetry:** A technique used to determine the concentration of colored compound(s) in solution.
- k. **Interfering substance:** A substance or quantity thereof that is not the measurand but that affects the result of the measurement.
- l. **IUPAC:** International Union of Pure and Applied Chemistry
- m. **LDT:** Laboratory –developed Test.
- n. **Linearity:** Linearity is the ability of a quantitative analytical method to provide results that are directly proportional to the concentrations of an analyte in test samples, within a given measuring interval. It is an important parameter to confirm when evaluating an analytical method because it verifies correct interpolation of results between points.
- o. **LMR:** Lower end of the measuring range is the lowest level at which defined conditions, including all stated characteristic of the method, are met.
- p. **LoB:** Limit of Blank is the highest value in a series of results on a sample that contains no analyte
- q. **LoD:** Limit of Detection is the lowest amount of analyte in a sample that can be detected with stated probability, although perhaps not quantified as an exact value.
- r. **LoQ:** When used without a prefix, the Limit of Quantitation is the lowest actual concentration at which an analyte is reliably detected and at which uncertainty of the test result is less than or equal to the goal set by the manufacturer or laboratory. The term may also be used with prefixes L for lower (LLOQ) and U for upper (ULOQ), respectively. Note: $LoB < LoD \leq LoQ$.
- s. **Matrix:** All components of a material system, except the analyte. A specimen matrix is the biological milieu in which an analyte exists (e.g., plasma, serum, urine, or other body fluids).

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

theranos	LDT Validation Report	Theranos Total Cholesterol Assay CL RPT-14040	Rev: 1
	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Description		Date: 09/25/2013	
Originator: Curtis Schneider			

- t. **Measuring Interval (reportable range; analytical measurement range or AMR):**
A measuring interval consists of all numeric values between the lower and upper numeric values for which a method can produce quantitative results suitable for clinical use. Where applicable, a linearity study is frequently used to establish or verify the measuring interval that can be reported for a measurement method. Alternatively, the lower limit of the measuring interval may be assigned as the LoQ (LLOQ).
- u. **Precision:** Precision is the closeness of agreement between indications or measured quantity values obtained by replicate measurements on the same or similar objects under specified conditions. It is usually expressed numerically in terms of standard deviation (SD) or percent Coefficient of Variation (%CV).
- v. **Reference interval:** The interval between and including two reference limits. It is common practice to define a reference limit so a stated fraction of the reference values is less than or equal, or greater than or equal, to the respective upper or lower limit.
- w. **SOP:** Standard Operating Procedure.
- x. **Spectrophotometry:** The quantitative measurement of the transmission (or reflection) properties of a material as a function of wavelength.
- y. **Testing System:** The entirety of the testing process, including instrument, sample, reagents, supplies, and procedures. Personnel are sometimes included in the definition.

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider	Date: 09/25/2013		

IV. Pre-clinical Validation

a. Analytical Measurement Range

i. Limits of Blank, Detection and Quantitation

The limits of blank, detection, and quantitation determined were 0.0 mg/dL, 6.6 mg/dL, and 92.9 mg/dL (93% recovery), respectively.

Limit of blank

CLSI guideline EP17-A section 4.3.1

Level	Number of samples	N	Mean	SD
Blank	1	20	0.0	0.0
Alpha	5%			
Parametric LoB	0.0			

Limit of detection

CLSI guideline EP17-A section 4.3.2

Level	Number of samples	N	Pooled SD
Low	1	20	3.9
Beta	5%		
Parametric LoD	6.6		

Limit of quantitation

CLSI guideline EP17-A section 5.1

Level	Number of samples	N
Low	1	20
Bias	-7.1	
Pooled imprecision	3.9	
95% total error	-14.8	
Allowable error	-	
LoQ	6.6	

Theranos Confidential

Page 7 of 13

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

TMP-00009 Rev. A, Released 08/01/13

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description		Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma	
Originator: Curtis Schneider		Date: 09/25/2013	

Level	Sample	n	Assigned value	Mean	Median	SD	CV
Blank	1	20	0	0.0	0.0	0.0	-
Low	1	20	100	92.9	94.0	3.9	4.2%

ii. Linearity

The Analytical Measurement Range (AMR) including linear measurement interval has been determined by Siemens. Refer to the **Analytical Range** section of the manufacturer product information insert for additional details.

b. Analytical Specificity

The analytical specificity for this assay was determined by observing the effect of bilirubin (10 mg/dL) on the recovery of total cholesterol (170 mg/dL) in a spiked plasma sample. No significant interference (NSI) was determined if the mean analyte concentration of an interferent-spiked sample reported within 10% of the mean analyte concentration of an un-spiked sample. Recovery of total cholesterol in the presence of bilirubin was 100% (see table below).

Analyte:	Interferent:	% Recovery
Total Cholesterol (mg/dL)	Bilirubin (mg/dL)	
170	10	100*

* NSI observed at interferent level tested.

c. Precision

Level = L1

Number of observations	80
Number of runs	40
Number of days	20
Runs per day	2
Replicates per run	2

Mean 103.6

	SD	95% CI	CV
Repeatability	2.4	2.0 to 3.1	2.4%
Between-run	0.9		0.9%
Between-day	3.2		3.1%
Within-laboratory	4.2	3.4 to 5.4	4.0%

Theranos Confidential

Page 8 of 13

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

TMP-00009 Rev. A. Released 08/01/13

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider	Date: 09/25/2013		

Level = L2

Number of observations	78
Number of runs	39
Number of runs excluded	1
Number of days	20
% of days with 1 run	5%
Runs per day	2
Replicates per run	2

CLSI guideline EP05-A2 section 10.4 recommends a minimum of 40 runs

Mean 174.5

	SD	95% CI	CV
Repeatability	2.5	2.0 to 3.1	1.4%
Between-run	0.0		0.0%
Between-day	4.1		2.3%
Within-laboratory	4.8	3.8 to 6.4	2.7%

Level = L3

Number of observations	80
Number of runs	40
Number of days	20
Runs per day	2
Replicates per run	2

Mean 265.9

	SD	95% CI	CV
Repeatability	4.2	3.4 to 5.4	1.6%
Between-run	0.0		0.0%
Between-day	2.6		1.0%
Within-laboratory	4.9	4.2 to 5.9	1.8%

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

TMP-00009 Rev. A, Released 08/01/13

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider	Date: 09/25/2013		

V. Clinical Validation

a. Method Comparison with Predicate (Accuracy/Comparability)

To test the accuracy of the assay on the Theranos System, 40 unique patient samples were screened on the predicate method (Siemens, Advia) and on the Theranos method. Using the predicate method eighteen (18) values were below the decision level of 200 mg/dL and twenty two (22) were above. Based on the results of the data examination, either a simple linear regression or alternative procedures were used to estimate expected (average) bias and the confidence interval of expected bias at the desired medical decision level(s) as per CLSI guidance EP09-A2. StatisPro was used for bias calculations. These estimates were compared with internal criteria to judge the acceptability of the Theranos method. Each sample was run in duplicate on the predicate, and the average used for comparison to the Theranos method. Some samples were stored before analysis on both methods. If the confidence interval for the predicted bias includes the defined acceptable bias or if the acceptable bias is greater than the higher limit of the confidence interval of the predicted bias, then the data do not show that the bias of the Theranos method is different from the acceptable bias or there is a high probability (97%) that the predicated bias is acceptable, respectively. The acceptable bias at each medical decision level was determined based on the total allowable error (TEa) minus the measured precision at the level closest to that decision level. Total allowable error (TEa) was taken from American Proficiency Institute (API) peer proficiency testing criteria or CLIA proficiency testing criteria for acceptable analytical performance, as printed in the Federal Register February 28, 1992;57(40):7002-186, when available. The TEa for Total Cholesterol is 10%. The table below shows the allowable bias and precision at 2 levels (values shown in parentheses) and the corresponding closest medical decision limits.

Table 1. Allowable Bias and Precision at the Medical Decision Levels

Medical Desirable Levels (mg/dL)	<200.0 (174.5)	>240.0 (265.9)
Precision (%)	2.3	1.0
Allowable Bias (%)	7.7	9.0

Theranos Confidential

Page 10 of 13

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

TMP-00009 Rev. A. Released 08/01/13

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider	Date: 09/25/2013		

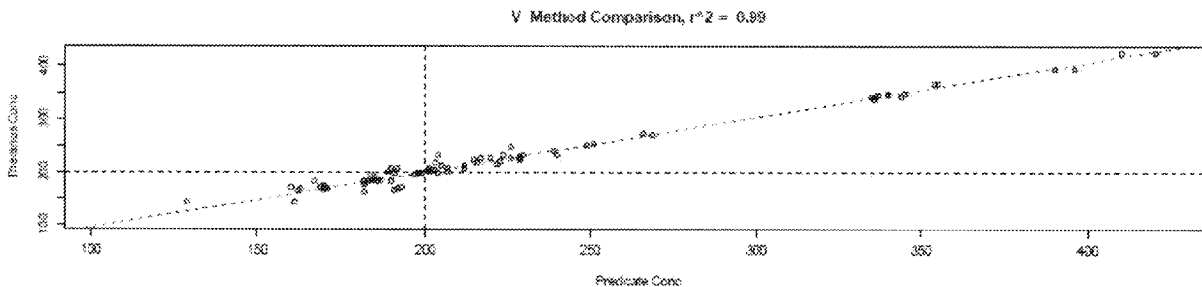


Figure 1. Graph showing Theranos method versus Predicate Method (Siemens Advia).

Simple linear regression was used to establish a slope, intercept and an r^2 . The slope, intercept and clinical correlation were determined to be 1.05, -11.99 and 0.99 respectively.

Comparability

CLSI guideline EP09-A2, IP section 7

Level ID	Value	Difference	SE	95% CI	Allowable difference
	90	-3.6	1.25	-6.1 to -1.0	5.4
	240	0.9	0.59	-0.3 to 2.1	14.4
	260	1.5	0.63	0.2 to 2.8	15.6

Difference is less than allowable bias: 6%.

The difference between the two methods is not greater than the allowable difference. The performance requirement is verified.

b. Transference and Verification of Reference Interval (Venous)

Reference interval for this analyte has been replaced by decision limits therefore verifying venous sample reference ranges is not required for Total Cholesterol.

Thirty two (32) new normal venous samples were tested, 31 (96.9%) reported values below the decision limit of 200 mg/dL and 1 (3.1%) reported a value above.

Theranos Confidential

Page 11 of 13

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

TMP-00009 Rev. A, Released 08/01/13

theranos	LDT Validation Report	Theranos Total Cholesterol Assay	Rev:
		CL RPT-14040	1
Description	Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider	Date: 09/25/2013		

c. Verification of Reference Interval with Finger Stick Samples

Verifying finger stick reference ranges not required for Total Cholesterol. Sixteen (16) new venous matched finger sticks samples were also tested, all 16 (100%) reported values below the decision limit of 200 mg/dL.

VI. Stability

a. Reagents

On-board Reagent Stability

System	Stability
ADVIA 1200	60 days
ADVIA 1650/1800	60 days
ADVIA 2400	60 days

For all systems, unopened reagents are stable until the expiration date printed on the product label when stored at 2° - 8°C. Do not freeze reagents.

For complete details, refer to the Methods Introduction section of the system-specific Operator's Guide.

b. Sample

Plasma samples for Total Cholesterol analysis are stable for 2 weeks at 2-8 °C, or at least 90 days at -20 °C.

c. Calibrators

The Siemens HDL/LDL Cholesterol Calibrator should be stored at 2-8 °C. Unopened the calibrator is stable until the expiration date on the vial label. After reconstitution the calibrator is stable for 3 days.

theranos	LDT Validation Report	Theranos Total Cholesterol Assay CL RPT-14040	Rev: 1
	Description: Validation Report for Theranos Laboratory Developed Assay of Total Cholesterol in Lithium Heparin Plasma		
Originator: Curtis Schneider		Date: 09/25/2013	

REVISION HISTORY			
Revision Level	Effective Date	Initiator	ECO Number
A	11/06/2013	A. Rosendorff	CL ECO-00117
Section Number	Description and Justification of Changes		
All	Initial Release		

Theranos Internal Only

Any retransmissions, reproductions, dissemination or other use of these materials by persons or entities other than the intended recipient is prohibited. This document supersedes all earlier or previous documents unless approved in writing.

TMP-00009 Rev. A, Released 08/01/13